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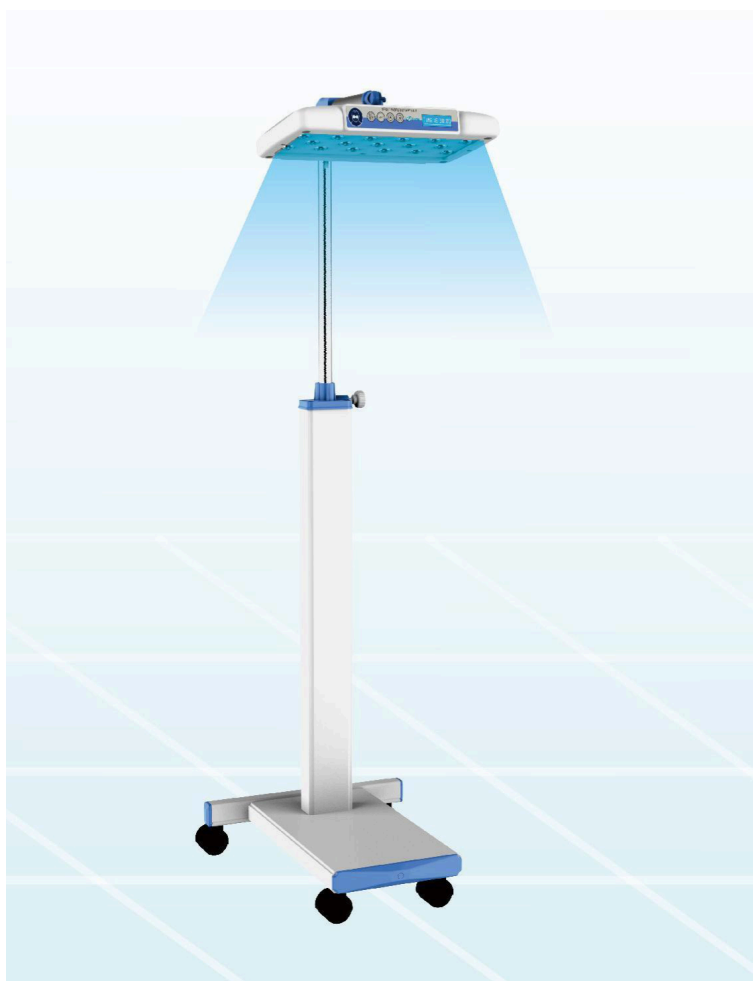
The **PCB-FL-01 Infant Phototherapy Unit** is a mobile LED phototherapy lamp designed for the treatment of neonatal jaundice (hyperbilirubinemia). It uses a bank of **17 blue LED bulbs (420–490 nm)** at the clinically effective wavelength, with a **50,000-hour service life** — replacing traditional fluorescent tubes that lose intensity quickly and need frequent service.

Irradiance intensity is adjustable in **3 grades (Low / Medium / High)** so the clinical team can match the dose to the bilirubin level. A built-in **LCD shows therapy time + integral time separately**, with **both count-up and count-down timers** for precise documentation and dose control.

The radiant head pivots **0–180°** and the column extends from **850 mm to 1640 mm**, so the unit pairs with any infant incubator, radiant warmer or baby cradle. **Fanless design** (no noise) keeps the newborn calm during treatment. Four braked castors on an aluminum-alloy + steel base keep the unit mobile and stable.

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Projetada para tratamento de icterícia neonatal.

17 × LÂMPADAS LED AZUIS

Bank of 17 high-intensity blue LEDs at **420–490 nm** — the clinically optimal wavelength for converting bilirubin. **50,000-hour** service life vs only ~2,000 hours for traditional fluorescent tubes.

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Adjustable irradiance from **1,450 to 4,500 $\mu\text{W}/\text{cm}^2$** — match the dose to the clinical protocol and bilirubin level. High mode delivers intensive phototherapy.

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Uniform high-intensity light across the effective surface area (**400 × 300 mm at 360 mm**) — every part of the newborn's exposed skin receives consistent dose.

TELA LCD + TIMERS DUPLOS

LCD shows therapy time and integral time **separately**. **Count-up** for documentation, **count-down** (30 min – 48 h) for protocol-driven dosing. Faixa tempo integral up to 99,999 h.

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LED design produces minimal heat — no cooling fan needed. **Zero fan noise**, no air movement near the newborn, no infection-control concerns from fan blade dust.

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Radiant head pivots **0–180°**, column adjusts from **850 to 1,640 mm** — pairs with any incubator, radiant warmer or baby cradle. Optimal positioning for every clinical setup.

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Aluminum-alloy + steel support base with **4 castors with brake** — easy to move between rooms, locks securely in place during treatment.

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50,000-hour Vida útil LED means **5+ years of continuous use** at typical clinical duty cycles. Far lower lifecycle cost than fluorescent or halogen alternatives.

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INCUBADORA NEONATAL

Position the radiant head above any closed incubator — PCB-INC-10 / 20 / 30 or any standard incubator. Adjustable height + angle.

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Pairs with open-care radiant warmers (PCB-IRW-10 / 20) for combined warming + phototherapy treatment.

BERÇO

Use over any newborn cradle, bassinet or nursery bed. Mobile design moves between cribs as needed.

BERÇÁRIO

Treat multiple newborns in the well-baby nursery — move the unit between cribs throughout the shift.

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Fabricado sob ISO 13485:2016

The PCB-FL-01 is manufactured under **ISO 13485:2016 · TÜV Rheinland** certification, with **CE 0197** mark for the European market. Every unit passes a full safety + functional test before shipping, ensuring compliance with regulatory requirements in every market.

✓ CE 0197

✓ ISO 13485:2016

✓ TÜV Rheinland

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Complete PCB-FL-01 data sheet in PDF, ready to print or send to your customer.

[Baixar folheto \(PDF\) ↓](#)

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PCB Medical · Kentucky (USA)

Industrial Park Av, Suite 30
Louisville, Kentucky 40041
Tel. (502) 333-0006

www.pcbmedical.net

PCB Medical · China

2103 Citic Tower, 1500
Xiangjiang North Road
Changsha, China