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The **PCB-IRW-10 Infant Radiant Warmer** is a microprocessor-based, servo-controlled open-care unit designed for delivery rooms, newborn resuscitation, neonatal recovery and procedural care where the clinical team needs immediate, unobstructed access to the newborn. The unit operates in **manual mode or automatic (servo-skin) mode**, switching between them with a single control.

An anti-explosion **micro-crystal quartz infrared radiation tube** delivers stable, uniform heat to the mattress surface, with **± 2 °C uniformity** across the bed. Set temperature and skin temperature are each shown on independent LED indicators, and a **second thermal cut-off** protects the newborn even if the primary sensor fails.

The Luz de observação LED has adjustable brightness for night-time work without disturbing the baby.

The bed includes **four transparent side protectors** with 2 selos de infusão, an integrated tray for accessories and an RS-232 connector for service / data export. The aluminum-alloy structural frame keeps the unit light and corrosion-resistant for daily hospital use.

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MODOS MANUAL + AUTOMÁTICO

Microprocessor-controlled unit allows for two modes: **manual** (operator sets power level directly) and **automatic** (servo-skin — system maintains the set skin temperature). One control switches between modes.

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Micro-crystal quartz infrared radiation tube — stable, uniform heat with anti-explosion construction for clinical safety. Long service life under continuous hospital use.

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Independent secondary thermal cut-out protects the newborn even if the primary temperature sensor fails — critical safety layer for radiant heat sources.

CONJUNTO COMPLETO DE ALARMES

Audible + visual alarms for: **power failure, temperature deviation, temperature sensor failure, skin sensor failure, over temperature.**

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Luz de observação LED with adjustable brightness — night-time work without disturbing the baby, no warm-up time, low heat output.

Four folded transparent side protectors with **2 selos de infusão** — keep the newborn safe while providing full access for IV lines, monitoring leads and procedures.

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Excellent detail

Heating head with metal grid

Metal grid protects the quartz tube from physical impact and contamination. Cleanable, durable, infection-control friendly.

COLUNA LATERAL

Siderail column

Integrated siderail column for IV-pole mounting, accessory rails and bracket attachment — flexible bedside configuration.

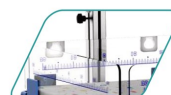
MATERIAL DE ALUMÍNIO

Aluminium alloy material

Aluminum-alloy structural frame — lighter than steel, corrosion-resistant, easy to clean. Long service life

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2 infusion seals

Two infusion seals in the side protectors — pass IV lines, ECG leads and oxygen tubing

under
continuous
hospital use.

without
breaking the
controlled
microclimate.

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O que vem na caixa.

Main body (Fonte radiante, Control system, Infant bed, Bracket) · **1 V-pole** · Skin temperature sensor · Tray · Mattress · Transparent protector · Castors · RS-232 cable.

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Fabricado sob ISO 13485:2016

The PCB-IRW-10 is manufactured under **ISO 13485:2016 · TÜV Rheinland** certification, with **CE 0197** mark for the European market. Every unit passes a full safety + functional test before shipping, ensuring compliance with regulatory requirements in every market.

✓ CE 0197

✓ ISO 13485:2016

✓ TÜV Rheinland

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Complete PCB-IRW-10 data sheet in PDF, ready to print or send to your customer.

[Baixar folheto \(PDF\) ↓](#)

**PCB Medical · Kentucky
(USA)**

Industrial Park Av, Suite 30
Louisville, Kentucky 40041
Tel. (502) 333-0006

www.pcbmedical.net

PCB Medical · China

2103 Citic Tower, 1500
Xiangjiang North Road
Changsha, China

© 2026 PCB Medical Equipment